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USER LEAFLET for German Health Care GmbH Urinary Drainage Systems

General information:

Dear user, please read all of the following information carefully before using one of the urinary drainage systems listed below. Specific details about the individual systems can be found in the following table.

Note:

We wish to point out that the products below may be used solely by physicians and trained specialists upon medical referral.

Uses / Description

Closed urinary drainage system for connection to sterile catheters. The products bearing US are suitable to be used with urinary condoms only.

Product	Filling volume in ml	Length of tube in cm	Sterility	Bed bag system	Leg bag system	Drainage tap	Can be used with	Pump ball	Fleece coating	Fixing	Fixing Combinations ¹			
Care Flow				closed	Number of chambers		Ca-theter Urinary condom	Rever-se	twi n hol- der	Cor- d	twi n hol- der	Velcro strap	Fixation stocking	¹ Articles suitable for being combined are not included with delivery. They can be ordered separately from us.
UB 4000 TUR	4000	120	x	x		x	x		x		x			
UB 4000	4000	120	x	x		x	x			x				
UB 2000	2000	120/90	x	x		x	x			x				
UB 2000 A	2000	110	x			x	x			x				
UB 2000 E	2000	90	x			x	x							
UB 1000 Day/Night	1000	30/60/90	x	x		x	x				x	x		
UB 750	750	10/50	x		1	x	x		x			x	x	
UB 600	600	10/50	x		3	x	x		x			x	x	
UB 500	500	10/50	x		3	x	x		x			x	x	

Indications /Uses

GHC Urinary Drainage Systems are used in patients with an indwelling urinary catheter, percutaneous renal fistula or urinary condom in order to collect urine. The type of patient care provided is to be determined according to individual needs. It is very important that any existing instructions given by the physician in charge are followed.

Contraindications and usage restrictions

There are no known absolute contraindications. Usage restrictions may apply depending on the clinical picture of the patient to be treated. Leg bag systems should not be used in patients with macrohaematuria. Our urinary drainage systems and tubes must not come into contact with areas of the skin with visible pathological changes.

Method of use / Procedure / Hygiene information

Whenever you wish to handle our urinary drainage systems (to change the system/drain off urine), please wash your hands thoroughly with soap and water beforehand and then use a hygienic hand disinfection product afterwards. Please use disposable gloves.

- Remove the bag from its sterile packaging
- Remove the protective cap from the cone and, if necessary, close the counter bead located at the drainage tap (if open).
- Separate the existing bag system from the catheter and disinfect the catheter funnel with a suitable disinfectant. Allow to take effect.
- Connect the bag system to the catheter by inserting the cone into the catheter funnel.
- Attach the bed bag system to the bed frame using the twin holder and cord.
- Leg bag: For optimum drainage, mount the bag 30 cm below the bladder.
- Please make sure not to bend the tubes on our urinary drainage

systems, particularly those on the leg bag, as this will otherwise affect drainage.

- The tube clamp and urine collection port integrated in the cone allow samples to be taken with a luer lock syringe without the need for cannulas and without causing any cannula injuries.

- Please make sure to open the tube clamp after taking a urine sample, as drainage will otherwise be prevented.

Emptying the system

- If necessary, take the drainage tap out of the doghouse. Hold it over a suitable collection receptacle and open the bag by moving the counter bead in the drainage tap. After fully emptying the system, slide the counter bead back and, if necessary, place the drainage tap in the doghouse for safe and hygienic storage.

Disposal

- In accordance with standard national regulations, the empty bag system is to be disposed of with household waste (non-recyclable waste).

Service life/Indwelling periods

- All of the urinary drainage systems above are for single use. In cases where a system has been removed from its original packaging and connected to a catheter or urinary condom, it MUST NOT, in so far as the articles have been separated from each other, be reconnected to a catheter. Closed urinary drainage systems should not be routinely changed at fixed intervals, but whenever the physician or nursing staff consider it necessary.

Packaging / Supply form

- Each article comes in sterile, individual packaging. Sterility is only ensured for as long as the packaging is unopened and undamaged, but for no longer than the date specified on the label.

Warnings / Precautions

Please ensure that any urinary drainage systems connected to an indwelling bladder catheter are not subjected to any tension whatsoever, as this will otherwise affect the location of the catheter and its drainage output (RISK OF INJURY)! If patients with cerebral dysfunction are to be provided with our urinary drainage systems, appropriate care/supervision must be ensured. This also applies to small children.

Disclaimer:

In principle, liability can only be accepted in relation to the connection/attachment of GHC's own products to the product you are using. Should you wish to connect/attach products made by other manufacturers, this may be done, but a written release from German Health Care GmbH pertaining to this product must first be obtained.

GHC accepts no liability for any injuries or other damage resulting from any use of this product, which is not in accordance with current GHC recommendations.

ATTENTION: all information has been provided to the best of our knowledge at the time of going to print. Subject to technical changes.

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INFORMAZIONI PER L'USO dei sistemi di drenaggio urinario German Health Care GmbH