



Intra – Uterine Insemination Catheter

Instruction for use

INDICATIONS

Used for artificial insemination.

HOW SUPPLIED:

The device is sterilized by ethylene oxide. Content is sterile, non-pyrogenic in unopened undamaged package. Do not use if package has been damaged or has been opened.

STORAGE

Store at 5°C to 30°C. Do not expose to organic solvents, ionizing radiation or ultraviolet light. Rotate inventory so that the articles are used prior to expiration date on the package label.

WARNING

- For single use only
- Do not use if any sign of product damage is visible
- Do not re-use, reprocess or re-sterilize. Reuse may lead to infection and Pyrogenicity. Reprocessing or Re-sterilization may damage the product.

PRECAUTIONS

Periodical Evaluation is advised.

INSTRUCTION OF USE

- Once the sperm sample is ready, the physician introduces a speculum to the vagina to visualise the cervix.
- A mild cleaning solution may be used to clean the cervix and the surrounding vaginal tissues.
- A small amount of the sperm is drawn in to the catheter by using the catheter with a syringe.
- The catheter is passed through the cervix.
- The sperm is injected in to the uterus.
- The catheter and the speculum will then be removed.



SPECIFICATIONS

- Available in 6Fr
- Available length 17cm
- Indwelling time less than 1 day

ARTICLE NUMBERS

IUI

PRODUCT SAFE DISPOSAL

Used products should be disposed in sanitary container to prevent possible contamination and cross infection

TABLE OF SYMBOLS

Symbol	Description
	Do not reuse
	Use by
	Lot Number
	Date of manufacture
	Sterilized using ethylene oxide
	Catalogue number
	Caution
	Manufacturer
	Temperature limitation
	Consult instruction for use
	Keep away from sunlight
	Keep dry
	Non pyrogenic
	Do not use if package is damaged
	CE Mark

Marflow AG
Soodstrasse 57
8134 Adliswil
Switzerland

 2265

June, 2016

Marflow AG, Soodstrasse 57, 8134 Adliswil, Switzerland
www.marflow.ch



Doc: IFU_IUI Catheter / v1.3