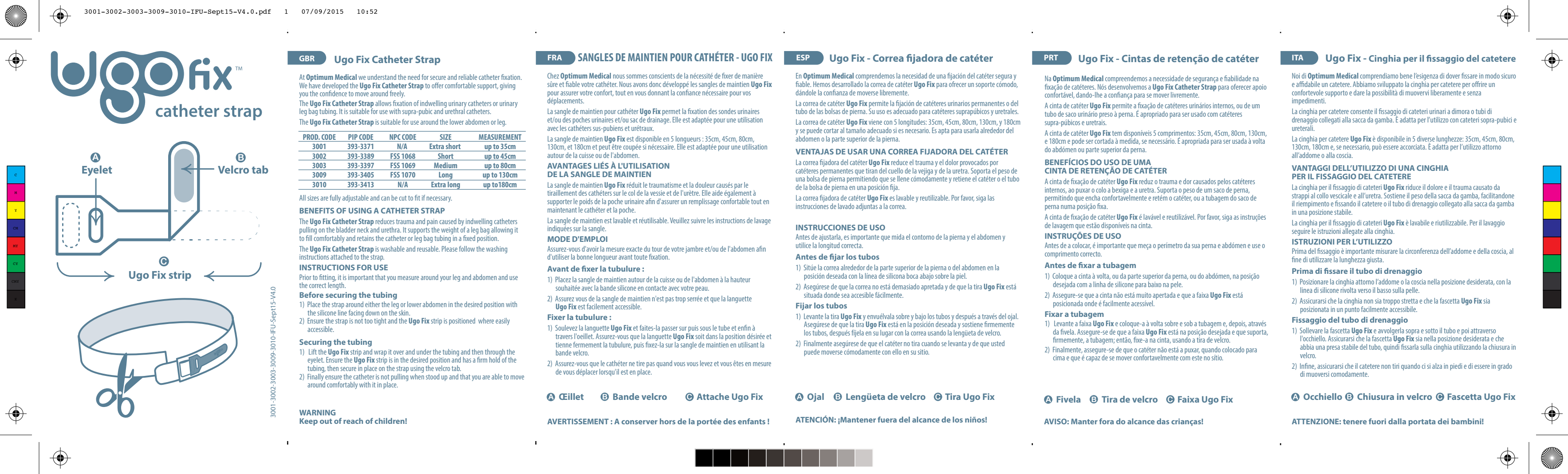
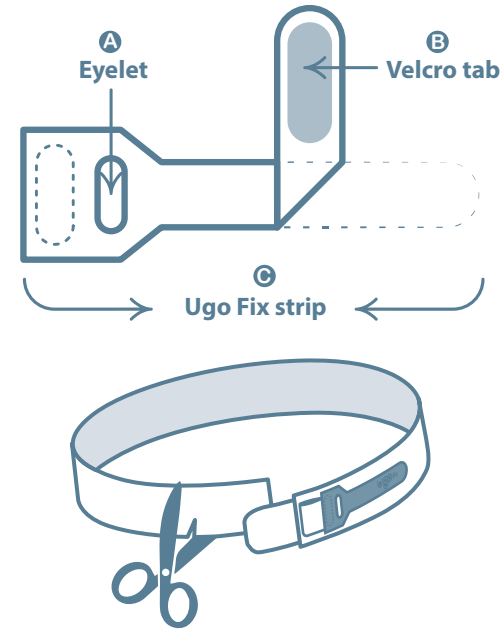
Ugo fix™ catheter strap

3001-3002-3003-3009-3010-IFU-Sept15-V4.0

GBR Ugo Fix Catheter Strap

At **Optimum Medical** we understand the need for secure and reliable catheter fixation. We have developed the **Ugo Fix Catheter Strap** to offer comfortable support, giving you the confidence to move around freely.

The **Ugo Fix Catheter Strap** allows fixation of indwelling urinary catheters or urinary leg bag tubing. It is suitable for use with supra-pubic and urethral catheters.

The **Ugo Fix Catheter Strap** is suitable for use around the lower abdomen or leg.

PROD. CODE	PIP CODE	NPC CODE	SIZE	MEASUREMENT
3001	393-3371	N/A	Extra short	up to 35cm
3002	393-3389	FSS 1068	Short	up to 45cm
3003	393-3397	FSS 1069	Medium	up to 80cm
3009	393-3405	FSS 1070	Long	up to 130cm
3010	393-3413	N/A	Extra long	up to 180cm

All sizes are fully adjustable and can be cut to fit if necessary.

BENEFITS OF USING A CATHETER STRAP

The **Ugo Fix Catheter Strap** reduces trauma and pain caused by indwelling catheters pulling on the bladder neck and urethra. It supports the weight of a leg bag allowing it to fill comfortably and retains the catheter or leg bag tubing in a fixed position.

The **Ugo Fix Catheter Strap** is washable and reusable. Please follow the washing instructions attached to the strap.

INSTRUCTIONS FOR USE

Prior to fitting, it is important that you measure around your leg and abdomen and use the correct length.

Before securing the tubing

- Place the strap around either the leg or lower abdomen in the desired position with the silicone line facing down on the skin.
- Ensure the strap is not too tight and the **Ugo Fix** strip is positioned where easily accessible.

Securing the tubing

- Lift the **Ugo Fix** strip and wrap it over and under the tubing and then through the eyelet. Ensure the **Ugo Fix** strip is in the desired position and has a firm hold of the tubing, then secure in place on the strap using the velcro tab.
- Finally ensure the catheter is not pulling when stood up and that you are able to move around comfortably with it in place.

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