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Instructions for Use

BARD® MAGNUM® Reusable Core Biopsy Instrument

BARD® MAGNUM® Disposable Core Biopsy Needle with Spacer

Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

A. General Information and Device Description:

The BARD® MAGNUM® Biopsy Instrument is a spring-loaded, reusable core tissue biopsy device. It has selectable penetration depths of 22mm and 15mm. The BARD® MAGNUM® Needle is a single patient use core biopsy needle designed exclusively for use with the BARD® MAGNUM® Reusable Core Biopsy Instrument for acquisition of core biopsy tissue samples.

B. How supplied:

- The MAGNUM® Instrument is sold nonsterile and is to be decontaminated and processed (e.g. cleaned, lubricated, etc.) prior to use by the end-user.
- The MAGNUM® Biopsy Needles are sold sterile for single patient use only and are available in various gauge sizes and lengths. The MAGNUM® Needles are supplied sterile and nonpyrogenic unless the package has been opened or damaged.

C. Indications for Use:

The MAGNUM® Biopsy System (instrument and needles) is intended for use in obtaining biopsies from soft tissues such as liver, kidney, prostate, breast, spleen, lymph nodes and various soft tissue tumors.

D. Contraindications:

Not intended for use in bone.

E. Warnings:

1. Good medical judgment should be exercised in considering biopsy on patients who are receiving anticoagulant therapy or who have a bleeding disorder.
2. The collection of multiple needle cores may help to ensure the detection of any cancer tissue.
3. A “negative” biopsy in the presence of suspicious radiographic findings does not preclude the presence of carcinoma.
4. Post-biopsy patient care may vary with the biopsy technique utilized and the individual patient’s physiological condition. Observation of vital signs and other precautions should be taken to avoid and/or treat potential complications that may be associated with biopsy procedures.
5. The MAGNUM® Disposable Core Biopsy Needle with spacer has been designed for single use only. Reusing this medical device bears the risk of cross-patient contamination as medical devices – particularly those with long and small lumina, joints, and/or crevices between components – are difficult or impossible to clean once body fluids or tissues with potential pyrogenic or microbial contamination have had contact with the medical device for an indeterminable period of time. The residue of biological material can promote the contamination of the device with pyrogens or microorganisms which may lead to infectious complications.
6. Do not resterilize the MAGNUM® Disposable Core Biopsy Needle with spacer. After resterilization, the sterility of the product is not guaranteed because of an indeterminable degree of potential pyrogenic or microbial contamination which may lead to infectious complications. Cleaning, reprocessing and/or resterilization of the present medical device increases the probability that the device will malfunction due to potential adverse effects on components that are influenced by thermal and/or mechanical changes.

F. Precautions:

1. Use only BARD® MAGNUM® Biopsy Needles with the BARD® MAGNUM® Biopsy Instrument. We cannot recommend the use of biopsy needles made by other manufacturers.
2. This product should be used by a physician who is completely familiar with the indications, contraindications, limitations, typical findings and possible side effects of core needle biopsy, in particular, those relating to the specific organ being biopsied.
3. The introduction of the needle into the body should be carried out under imaging control (ultrasound, X-Ray, CT, etc.).
4. Never test the instrument while the needle is installed in the instrument. This could result in needle damage and/or patient/user injury.
5. Unusual force applied to the stylet or unusual resistance against the stylet, while extended out of the supportive cannula may cause the stylet to bend at the specimen notch. A bent specimen notch may interfere with needle function.
6. Inspect the instrument for signs of deterioration or damage (cracking, blistering, separation of coating, pitting). DO NOT USE if deterioration or damage is observed.
7. After use, the MAGNUM® Needles may be a potential biohazard. Handle and dispose of in accordance with acceptable medical practice and applicable local, state, and federal laws and regulations.

G. Potential Complications:

Potential complications associated with core biopsy procedures are site specific and include, but are not limited to: hematoma; hemorrhage; infection; adjacent tissue injury; pain; bleeding; hemoptysis; hemothorax; non-target tissue, organ or vessel perforation; pneumothorax; and air embolism. Air embolism is a rare but serious potential complication of lung biopsy procedures. Rapid deterioration of neurological status and/or cardiac arrhythmia may be indicative of air embolism. Prompt diagnosis and treatment must be considered if the patient exhibits signs or symptoms of air embolism.

H. Equipment Required:

- Appropriate imaging modality accessories
- Surgical gloves and drapes
- Local anesthetic
- BARD® TRUGUIDE® Coaxial Cannula (optional)
- Scalpel
- Sample collection container
- Other equipment as necessary

I. Directions for Use:

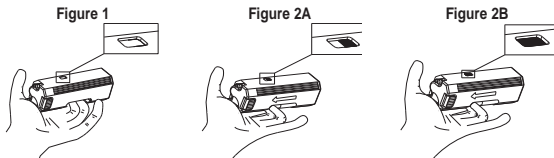
BARD® MAGNUM® Biopsy Instrument preparation:

After following the directions for reprocessing (Reference section J):

1. Using aseptic technique, remove the instrument from its package.
2. We suggest cradling the instrument in your palm with your fingers on the white cocking slide (See Figure 1). Energize (cock) the instrument by pulling back with full pressure on the white cocking slide twice until both sleds are fully engaged. Notice the status indicator is now red. (See Figures 2A and 2B.)
3. Test the instrument by first moving the safety lever from "S" (Safe) to "F" (Fire, Ready) then depress the trigger button to actuate the instrument.
4. Re-energize (cock) the instrument.

WARNING: WHEN THE SAFETY LEVER IS IN THE "F" POSITION, THE INSTRUMENT IS READY TO FIRE. CARE SHOULD BE TAKEN NOT TO INADVERTENTLY DEPRESS THE TRIGGER AND FIRE THE DEVICE.

5. Select penetration depth of 22mm or 15mm by moving the lever in the rear of the instrument to the appropriate setting.

BARD® MAGNUM® Biopsy Needle preparation:

The needle has centimeter markings for depth measurement and is packaged with a spacer. The spacer holds the relative position of the stylet and cannula during insertion either into the patient or into the BARD® MAGNUM® Instrument.

Prior to use, determine the appropriate gauge and length of the needle required for the specific biopsy to be performed. Using aseptic technique, remove the needle from the package and from the protective sheath.

Precaution: Before using, inspect the needle components for damage. **DO NOT USE** if the needle components are damaged. **Note:** The tip is made with a slight back bend. This is not a defect.

Biopsy Procedure:

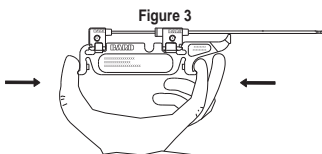
The biopsy procedure must be performed using appropriate aseptic technique.

1. Prepare the site as required. Adequate anesthesia should be administered prior to incision of the skin.

Recommendation: For ease of insertion, puncture the skin with a scalpel at the entry site.

2. Positioning:

a) With the BARD® MAGNUM® Biopsy Instrument and Biopsy Needle: Open the cover of the energized (cocked) instrument. Align the holes on the needle hubs with the posts of the instrument carrier sleds. Partially close the cover to maintain the position of the needle hubs. Compress the ends of the spacer to release and remove (see Figure 3). Close the cover. Introduce the needle, under imaging control where appropriate, through the incision until the needle point is proximal to the area to be biopsied.

OR

b) With ONLY the BARD® MAGNUM® Biopsy Needle with Spacer:

Introduce the needle with the unique spacer attached under imaging control through the incision until the needle point is proximal to the area to be biopsied. When the position is confirmed, attach the energized (cocked) BARD® MAGNUM® biopsy Instrument. Taking care to maintain the needle orientation, align the holes on the needle hubs with the posts of the instrument carrier sleds. Partially close the cover to maintain the position of the needle hubs. Compress the ends of the spacer to release and remove (see **Figure 3**). Close the cover.

- While maintaining the instrument position and needle orientation, move the safety lever from "S" (Safe) to "F" (Fire, Ready). Depress the trigger button to cause both the stylet and cannula to automatically advance.
- Remove the instrument and needle from the patient.

Note: It is not necessary to take the needle out of the instrument to retrieve the core tissue specimen.

- Expose the sample by pulling back the white cocking slide ONCE. Notice the status indicator is one-half silver, one-half red and the sample notch is exposed. Remove the sample.
- Energize (cock) the instrument by pulling back the white cocking slide one more time. The status indicator is now red. Take additional biopsies if required. If additional biopsies are NOT being taken, then remove and discard the needle, release the safety and fire the instrument. The status indicator is now silver indicating the instrument is not energized.

Note: If collecting multiple samples, inspect the needle for damaged point, bent shaft or other imperfections after each sample is collected. Do not use the needle if any imperfection is noted.

BARD® recommends the MAGNUM® Instrument be cleaned and lubricated prior to every use. End-users may sterilize the instrument after each cleaning and lubrication.

J. Directions for Reprocessing MAGNUM® Instrument:

Automated Cleaning Method and Thermal Disinfection Instructions

- DO NOT use 2% glutaraldehyde or other bactericidal solutions.
- DO NOT use highly alkaline or highly acidic cleaners.
- DO NOT clean with concentrated chlorhexidine gluconate or any other agent containing alcohol.
- DO NOT sterilize by irradiation.
- DO USE appropriate personal protective equipment as dictated by facility protocol.

STEP	AUTOMATED CLEANING METHOD AND THERMAL DISINFECTION INSTRUCTIONS
Point of Use	No particular requirements.
Containment and Transportation	No particular requirements.
Instrument Preparation	<p>Note: To facilitate reprocessing, the MAGNUM® Biopsy Instrument must be in the 22mm position with the lid open. This configuration provides optimal access to the internal areas for cleaning.</p> <p>Remove and discard the needle assembly. Further disassembly of the MAGNUM® Instrument is not recommended.</p>
Enzymatic Presoak	<p>Prepare enough multi-tiered enzymatic cleaner solution (e.g. STERIS® EnzyCARE 2®), following the manufacturer's instructions, to completely cover the MAGNUM® Instrument.</p> <p>Allow the instrument to soak in the enzymatic cleaning solution for a minimum of 5 minutes.</p> <p>Use a sterile syringe to introduce the cleaning solution to the hard-to-reach areas (e.g. crevices, threads) of the instrument.</p> <p>Actuate the instrument while in the cleaning solution to ensure the cleaning solution reaches the hard-to-reach areas.</p>

STEP	AUTOMATED CLEANING METHOD AND THERMAL DISINFECTION INSTRUCTIONS
Rinse	<p>Prepare a rinse bath with cool tap water to completely cover the instrument.</p> <p>With the lid open, rinse the instrument by a) agitating the instrument under the water level, b) bringing the instrument above the water level, and c) re-immersing the instrument under the water level. Repeat this process until there is no visible evidence of the cleaning solution.</p> <p>Actuate the instrument while in the rinse water to ensure the rinse water reaches the hard-to-reach areas.</p> <p>Note: The instrument should not be rinsed under running water as this may cause splashing and contamination of user.</p>
Manual Cleaning	<p>Prepare enough manual cleaning solution (e.g. STERIS® Manu-Klenz®) in accordance with its manufacturer's instructions to completely cover the instrument.</p> <p>Place the instrument into the manual cleaning solution.</p> <p>While holding the instrument under the solution level and using a soft-bristle brush, brush all accessible areas of the instrument until there is no visible evidence of debris or soil.</p> <p>Actuate the instrument while under the cleaning solution to ensure the cleaning solution reaches the hard-to-reach areas of the instrument.</p> <p>Use a sterile syringe to introduce the manual cleaning solution to the hard-to-reach areas (e.g. crevices, threads) of the instrument.</p> <p>Repeat all steps until there is no visible evidence of debris or soil.</p>
Rinse	<p>Prepare a rinse bath with cool tap water to completely cover the instrument.</p> <p>With the lid open, rinse the instrument by a) agitating the instrument under the water level, b) bringing the instrument above the water level, and c) re-immersing the instrument under the water level.</p> <p>Actuate the instrument while in the rinse water to ensure the rinse water reaches the hard-to-reach areas.</p> <p>Repeat rinsing until there is no visible evidence of the cleaning solution.</p> <p>Note: The instrument should not be rinsed under running water as this may cause splashing and contamination of user.</p>
Ultrasonic Cleaning	<p>Fill an ultrasonic cleaning unit with a fresh multi-tiered enzymatic cleaning solution prepared by following the manufacturer's instructions using processed water (e.g. deionized water). The enzymatic cleaning solution should be enough to completely cover the instrument.</p> <p>Make sure the lid of the instrument is open and the instrument's internal mechanisms are facing upward.</p> <p>Sonicate the instrument for a minimum of 15 minutes at the frequency of 40 kHz.</p> <p>Use a sterile syringe to introduce the cleaning solution into the hard-to-reach areas of the instrument.</p> <p>Actuate the instrument while under the cleaning solution to ensure the cleaning solution reaches the hard-to-reach areas of the instrument.</p>
Rinse	<p>Prepare a rinse bath with cool tap water to completely cover the instrument.</p> <p>With the lid open, rinse the instrument by a) agitating the instrument under the water level, b) bringing the instrument above the water level, and c) re-immersing the instrument under the water level.</p> <p>Actuate the instrument while in the rinse water to ensure the rinse water reaches the hard-to-reach areas.</p> <p>Repeat rinsing until there is no visible evidence of the cleaning solution.</p> <p>Note: The instrument should not be rinsed under running water as this may cause splashing and contamination of user.</p>

STEP	AUTOMATED CLEANING METHOD AND THERMAL DISINFECTION INSTRUCTIONS																								
Washer-Disinfector Cleaning and Thermal Disinfection	<p>With the lid open, place the instrument in the washer-disinfector with the internal mechanisms facing the ceiling of the washer-disinfector. Take care to not place other instruments on the MAGNUM® Instrument.</p> <p>Process the instrument through to the following cycle:</p>																								
	<table border="1"> <thead> <tr> <th>Phase</th> <th>Recirculation Time Minutes</th> <th>Water Temperature</th> <th>Detergent Type Concentration (If applicable)</th> </tr> </thead> <tbody> <tr> <td>Pre-Wash</td> <td>05:00</td> <td>Cold Tap Water: 61°F (16°C) Maximum</td> <td>N/A</td> </tr> <tr> <td>Enzyme Wash</td> <td>05:00</td> <td>Hot Tap Water: 109°F (43°C) Minimum</td> <td>Enzymatic Cleaner</td> </tr> <tr> <td>Wash 1</td> <td>10:00</td> <td>140°F (60°C) Set Point</td> <td>Neutral pH Cleaner</td> </tr> <tr> <td>Rinse 2</td> <td>05:00</td> <td>Hot Tap Water: 109°F (43°C) Minimum</td> <td>N/A</td> </tr> <tr> <td>Thermal Disinfection</td> <td>05:00</td> <td>Hot Tap Water: 185°F (85°C) Minimum</td> <td>N/A</td> </tr> </tbody> </table>	Phase	Recirculation Time Minutes	Water Temperature	Detergent Type Concentration (If applicable)	Pre-Wash	05:00	Cold Tap Water: 61°F (16°C) Maximum	N/A	Enzyme Wash	05:00	Hot Tap Water: 109°F (43°C) Minimum	Enzymatic Cleaner	Wash 1	10:00	140°F (60°C) Set Point	Neutral pH Cleaner	Rinse 2	05:00	Hot Tap Water: 109°F (43°C) Minimum	N/A	Thermal Disinfection	05:00	Hot Tap Water: 185°F (85°C) Minimum	N/A
	Phase	Recirculation Time Minutes	Water Temperature	Detergent Type Concentration (If applicable)																					
	Pre-Wash	05:00	Cold Tap Water: 61°F (16°C) Maximum	N/A																					
	Enzyme Wash	05:00	Hot Tap Water: 109°F (43°C) Minimum	Enzymatic Cleaner																					
	Wash 1	10:00	140°F (60°C) Set Point	Neutral pH Cleaner																					
	Rinse 2	05:00	Hot Tap Water: 109°F (43°C) Minimum	N/A																					
Thermal Disinfection	05:00	Hot Tap Water: 185°F (85°C) Minimum	N/A																						
Visual Inspection	<p>Inspect the instrument to ensure that all debris or soil has been removed. Inspection should be performed in a well-lit area.</p> <p>If there is any visible evidence of debris or soil, repeat all the aforementioned steps until there is no visible evidence of debris or soil remaining.</p>																								

Manual Cleaning Method

STEP	MANUAL CLEANING WITH SONICATION INSTRUCTIONS
Point of Use	No particular requirements.
Containment and Transportation	No particular requirements.
Instrument Preparation	<p>Note: The MAGNUM® Biopsy Instrument must be in the 22mm position as this configuration provides optimal access to the internal areas for cleaning.</p> <p>Remove and discard the needle assembly. Further disassembly of the MAGNUM® Instrument is not recommended.</p>
Initial Rinse	<p>Prepare a rinse bath with cool tap water.</p> <p>Note: The MAGNUM® Biopsy Instrument must be in the 22mm position as this configuration provides optimal access to the internal areas for cleaning.</p> <p>Remove and discard the needle assembly. Further disassembly of the MAGNUM® Instrument is not recommended.</p> <p>With the lid open, rinse the instrument by a) agitating the instrument under the water level, b) bringing the instrument above the water level, and c) re-immersing the instrument under the water level. Repeat this process until there is no visible evidence of the cleaning solution.</p> <p>Note: The instrument should not be rinsed under running water as this may cause splashing and contamination of user.</p>

STEP	MANUAL CLEANING WITH SONICATION INSTRUCTIONS
Enzymatic Presoak	Using warm tap water, prepare enough multi-tiered enzymatic cleaner (e.g. Ruhof® Endozime® AW Plus or Getinge® Renuzyme WR Enzymatic Cleaner), following the manufacturer's instructions, to completely cover the MAGNUM® instrument. Allow the MAGNUM® instrument with the lid open to soak in the cleaner for a minimum of 30 minutes at ambient temperature.
Manual Cleaning	<p>While holding the instrument under the solution level and using a soft-bristle brush, brush all accessible surfaces inside and out for a minimum of one minute with the enzymatic solution to remove blood and/or debris. Pull and hold the white cocking slide back once and brush the inside of the device for one additional minute to ensure the brush reaches all accessible areas inside the device.</p> <p>While moving the white cocking slide back and forth, use a sterile syringe filled with the enzymatic cleaning solution and flush hard-to-reach areas such as threads and crevices with the enzymatic cleaning solution.</p>
Rinse	<p>Prepare a rinse bath with cool tap water to completely cover the instrument.</p> <p>With the lid open, rinse the instrument by a) agitating the instrument under the water level, b) bringing the instrument above the water level, and c) re-immersing the instrument under the water level. Repeat this process until there is no visible evidence of the cleaning solution.</p> <p>Actuate the instrument while in the rinse water to ensure the rinse water reaches the hard-to-reach areas.</p> <p>Note: The instrument should not be rinsed under running water as this may cause splashing and contamination of user.</p>
Sonication	<p>Fill a sonicator with fresh multi-tiered enzymatic cleaner solution prepared by following the manufacturer's instructions using warm processed water (e.g. deionized water). Ensure there is enough water in the sonicator unit to come into contact with the rack/tray once it is inserted into the sonicator. Take care to not place other instruments on the MAGNUM® Instrument.</p> <p>Place rack/tray in sonicator and turn on the sonicator for a minimum of 30 minutes before inserting the instrument into the bath (per manufacturer's instructions for degassing).</p> <p>Ensure the lid of the instrument is open and the device is facing upward. Place the instrument into the rack/tray.</p> <p>Sonicate the instrument for a minimum of 30 minutes.</p>
Final Rinse	<p>After sonication, prepare a rinse bath with cool tap water.</p> <p>With the lid open, rinse the instrument by a) agitating the instrument under the water level, b) bringing the instrument above the water level, and c) re-immersing the instrument under the water level. Repeat this process until there is no visible evidence of the cleaning solution.</p> <p>Actuate the instrument while in the rinse water to ensure the rinse water reaches the hard-to-reach areas.</p> <p>Note: The instrument should not be rinsed under running water as this may cause splashing and contamination of user.</p>
Inspection	<p>Inspect the instrument to ensure that all debris or soil has been removed.</p> <p>Inspection should be performed in a well-lit area.</p> <p>If there is any visible evidence of debris or soil, repeat all the aforementioned steps until there is no visible evidence of debris or soil remaining.</p>

Lubrication:

BARD® recommends the MAGNUM® Instrument be cleaned and lubricated after every use to enhance the performance and longevity of the instrument. Silicone free, quality medical grade lubrication compatible with steam sterilization should be used to lubricate the MAGNUM® Instrument (e.g. Aesculap® Sterilit® I Oilspray JG 600, Miltex® Spray Lube 3-700, Weck®-Lube Concentrate #850150). Refer to the manufacturer's instructions for the use of the selected lubricating agent. Ensure all moving parts of the MAGNUM® Instrument are lubricated.

Steam Sterilization Instructions:**Precautions:**

- Remove and discard the needle assembly. Further disassembly of the MAGNUM® Instrument is not recommended.
- **DO NOT** use 2% glutaraldehyde or other bacterial solutions.
- **DO NOT** use highly alkaline or highly acidic cleaners.
- **DO NOT** clean with concentrated chlorhexidine gluconate or any other agent containing alcohol.
- **DO NOT** sterilize by irradiation.
- **DO USE** appropriate personal protective equipment as dictated by facility protocol.
- Prior to sterilization, the MAGNUM® Biopsy Instrument should be properly cleaned, lubricated and packed in accordance with these Instructions For Use.
- For sterilization, the MAGNUM® Biopsy Instrument must be in the 22mm position and not cocked/energized.
- Wherever applicable, the instrument should be packed into two sterilization pouches prior to sterilization.
- Biological indicators and/or steam integrators should be used within the sterilizer load to ensure the proper sterilization conditions of time, temperature, and saturated steam have been met.
- To ensure effectiveness, sterilization processes should be properly validated and monitored with the proper biological controls.

Any of the following nine sterilization cycle options may be used to sterilize the MAGNUM® Biopsy Instrument.

GRAVITY DISPLACEMENT OPTIONS	FULL CYCLE TEMPERATURE	FULL CYCLE EXPOSURE TIME	DRY-TIME (In chamber)	COOL DOWN TIME AT ROOM/AMBIENT TEMPERATURE
1	250°F (121°C)	40 minutes	45 minutes	30 minutes
2	270°F (132°C)	15 minutes	45 minutes	30 minutes
3	275°F (135°C)	10 minutes	30 minutes	30 minutes
GRAVITY DISPLACEMENT FLASH OPTION	270°F (132°C)	10 minutes	N/A	N/A

PRE-VACUUM OPTIONS	FULL CYCLE TEMPERATURE	FULL CYCLE EXPOSURE TIME	DRY-TIME (In chamber)	COOL DOWN TIME AT ROOM/AMBIENT TEMPERATURE
1	270°F (132°C)	4 minutes	30 minutes	30 minutes
2	273°F (134°C)	5 minutes	30 minutes	30 minutes
3	273°F (134°C)	18 minutes	30 minutes	30 minutes
4	275°F (135°C)	3 minutes	20 minutes	30 minutes
PRE-VACUUM FLASH OPTION	270°F (132°C)	4 minutes	N/A	N/A

Note: The aforementioned reprocessing steps were validated and are recommended in accordance with applicable industry standards such as AAMI TIR 12, AAMI TIR 30, and ISO 17664.

For additional information on cleaning, lubrication and sterilization, or technical assistance, please call Bard's Medical Services and Support at 1-800-562-0027 (Outside U.S.: 01-770-784-6704).

TROUBLESHOOTING GUIDELINE

PROBLEM	TRY THIS	IF THIS DOES NOT WORK
<ul style="list-style-type: none"> Instrument hard to energize or does not energize completely. 	<ul style="list-style-type: none"> Position instrument in hand correctly (Section I). Pull cocking slide all the way back (Section I). Ensure that safety is not in "F" position (Section I). Clean and lubricate instrument (Section J). 	<ul style="list-style-type: none"> Return to Bard for service. Call inside U.S. 1-800-526-4455, ext. 9-6266. Outside U.S. 01-770-784-6266, for instructions on returning the product.
<ul style="list-style-type: none"> Instrument hard to fire or does not fire. 	<ul style="list-style-type: none"> Ensure that instrument is completely energized (Section I). Ensure that safety is in "F" position (Section I). Clean and lubricate instrument (p.Section J). 	<ul style="list-style-type: none"> Return to Bard for service. Call inside U.S. 1-800-526-4455, ext. 9-6266. Outside U.S. 01-770-784-6266, for instructions on returning the product.
<ul style="list-style-type: none"> Instrument does not fire completely. 	<ul style="list-style-type: none"> Clean and lubricate instrument (Section J). 	<ul style="list-style-type: none"> Return to Bard for service. Call inside U.S. 1-800-526-4455, ext. 9-6266. Outside U.S. 01-770-784-6266, for instructions on returning the product.
<ul style="list-style-type: none"> Cannot switch between 15mm and 22mm positions. 	<ul style="list-style-type: none"> Ensure that instrument is completely energized (Section I). Clean and lubricate instrument (Section J). 	<ul style="list-style-type: none"> Return to Bard for service. Call inside U.S. 1-800-526-4455, ext. 9-6266. Outside U.S. 01-770-784-6266, for instructions on returning the product.

Warranty:

Bard Peripheral Vascular warrants to the first purchaser of this product that this product will be free from defects in materials and workmanship for a period of one year from the date of first purchase and liability under this limited product warranty will be limited to repair or replacement of the defective product, in Bard Peripheral Vascular's sole discretion or refunding your net price paid. Wear and tear from normal use or defects resulting from misuse of this product are not covered by this limited warranty.

TO THE EXTENT ALLOWABLE BY APPLICABLE LAW, THIS LIMITED PRODUCT WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT WILL BARD PERIPHERAL VASCULAR BE LIABLE TO YOU FOR ANY INDIRECT, INCIDENTAL OR CONSEQUENTIAL DAMAGES RESULTING FROM YOUR HANDLING OR USE OF THIS PRODUCT.

Some states/countries do not allow an exclusion of implied warranties, incidental or consequential damages. You may be entitled to additional remedies under the laws of your state/country.

An issue or revision date and a revision number for these instructions are included for the user's information on the last page of this booklet. In the event 36 months have elapsed between this date and product use, the user should contact Bard Peripheral Vascular to see if additional product information is available.

BARD® MAGNUM® Biopsy Needles assembled in Mexico.

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Bard and Magnum are trademarks and/or registered trademarks of C. R. Bard, Inc. or an affiliate. All other trademarks are the property of their respective owners.
Copyright © 2012 C. R. Bard, Inc. All rights reserved.
Copyright ©2012 C. R. Bard, Inc. Tous droits réservés.
Copyright ©2012 C. R. Bard, Inc. Alle Rechte vorbehalten.
Copyright ©2012 C. R. Bard, Inc. Tutti i diritti riservati.
Copyright ©2012, C. R. Bard, Inc. Reservados todos los derechos.